Français

ANI Sensor V2

Quantité: 25

Ref: ANI-SENS-V2

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Ref: ANI-SENS-V2

Indications for Use

The ANI Sensor V2 is a disposable, pre-gelled 3-lead array. Place the large and the small electrodes on the patient in order to detect a cardiac signal of good amplitude. For example, the large electrode can be put on the right shoulder and the small electrode at the apex of the heart as shown in the diagram on the blister label. The sensor is self-adhesive and contains bio-compatible materials where the array comes in contact with the skin.

The sensor is designed for use in children and adult population.

The ANI Sensor V2 is to be used with the ANI Monitor and may be worn for a continuous period of 24 hours in a hospital environment.

Cautions

CAUTION: Rx only – Federal law restricts this device to sale by or on the order of a physician.

ANI Sensor V2 should only be used by or in consultation with a health care provider familiar with their proper placement and use.

ANI Sensor V2 may damage the skin if removed carelessly.

ANI Sensor V2 should be applied only to intact, clean skin (e.g. not over open wounds, lesions, infected, or inflamed areas).

Ensure the skin area is clean, dry and hair free before sensor application. Use alcohol wipes for cleaning the application area.

Apply the ANI Sensor V2 within 1hr of removal from the blister.

Reddening of the skin and slight irritation may occur. This is normal.

Dispose of the ANI Sensor V2 properly after use under hospital guidelines.

The sensor is not waterproof.

Store in a cool dry place.

The ANI Sensor V2 is not made with natural rubber latex.

Precautions

During surgical procedures, ANI Sensor V2 should be placed as far as possible from any electro-surgical area to minimize unwanted RF current flow.

Replace the ANI Sensor V2 if it no longer sticks firmly to the skin.

Do not wear the ANI Sensor V2 during an MRI or X-Ray.

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